



[Knowledgebase](#) > [Using the Care Tracker](#) > [How can I see where the user is right now?](#)

## How can I see where the user is right now?

Mari Opstad - 2026-02-11 - [Comments \(0\)](#) - [Using the Care Tracker](#)

## How can I see where the user is right now?

### In short:

You can see the watch's location directly in the **Care Tracker app**.

When you activate **Find GPS**, a map appears showing the watch's real-time position.

### Here's how:

1. Open the **Care Tracker app**.
2. Select the watch you want to view.
3. Tap **Find GPS**.  
The map will open automatically, showing the watch's location.  
The watch sends its position every **15 seconds**, allowing you to follow the movement live.
4. When you're done, close **Find GPS** again.  
The watch will automatically return to normal updates every **10 minutes** to save battery.

### Tip:

- You can zoom in on the map and tap the marker to see the exact address.
- Check the timestamp at the bottom of the screen to see when the location was last updated.

### Note:

If the watch is shown as *offline*, the map will not appear until the connection is restored.