



[Knowledgebase](#) > [Setup and customization](#) > [How do I add more alarmreceivers?](#)

How do I add more alarmreceivers?

Mari Opstad - 2026-02-11 - [Comments \(0\)](#) - [Setup and customization](#)

How do I add more alarmreceivers?

In short:

You can easily add multiple alarmreceivers in the **Care Tracker app** so that several relatives receive a notification if the watch leaves a specific area.

Here's how:

1. Open the **Care Tracker app**.
2. Select the watch you want to add alert recipients to, and tap **Settings**.
3. Choose **Administrate alarmreceivers**.
4. Tap **Add alarmreceiver**.
5. Enter the name, phone number, and email address of the person you want to add.
6. Tap **Save**.
7. The person will receive an email with a link to create their account in the Care Tracker app.
8. Repeat the process if you want to add more alarmreceivers.

Tip:

- You can have several alarmreceivers for the same watch - for example, family members or other close contacts.
- Make sure all alarmreceivers have notifications enabled so they receive alerts if the watch leaves a geofence.

Note:

Only people you actively add as alarmreceivers can view the watch's position and receive notifications.

Related Content

- [Multiple caregivers. Can we be more than one in the app?](#)
- [How do I receive alarms?](#)