



[Knowledgebase](#) > [Setup and customization](#) > [How do I change notification settings for a watch?](#)

How do I change notification settings for a watch?

Mari Opstad - 2025-11-26 - [Comments \(0\)](#) - [Setup and customization](#)

How do I change notification settings for a watch?

In short:

You can customize how the Care Tracker app sends **notifications for each watch**. This means you decide how you're alerted when something important happens – for example, if the watch leaves an area or the battery is low.

Here's how:

1. Open the **Care Tracker app**.
2. Select the watch you want to change settings for, and tap **Settings**.
3. Choose **Notification settings**.
4. Here, you can adjust the following:
 1. **Enable or disable vibration**. Choose whether your phone should vibrate when a notification arrives.
 2. **Enable or disable sounds**. Choose whether notifications should include sound.
 3. **Alarm sound**. Select which sound should play for alarms, e.g. when the watch moves outside a geofence.
 4. **Warning sound**. Select a sound for warnings, e.g. when the watch has a low battery.

Tip:

- Changes are saved automatically when you adjust the settings.
- You can customize notifications for each watch individually if you have multiple Care Trackers.
- Choose clear sounds so you can easily distinguish between alarms and warnings.

Note:

If you're not receiving notifications, check that the Care Tracker app has permission to send notifications in your phone's settings.