



[Knowledgebase](#) > [Setup and customization](#) > [How do I create a geofence?](#)

How do I create a geofence?

Mari Opstad - 2026-02-11 - [Comments \(0\)](#) - [Setup and customization](#)

How do I create a geofence?

In short:

You can easily add a geofence in the **Care Tracker app** to receive notifications if the watch moves outside a specific area.

Here's how:

1. Open the **Care Tracker app**.
2. Select the watch you want to create a geofence for, and tap **Settings**.
3. Choose **Geofence** from the menu.
4. Tap **Add geofence**.
5. Give the geofence a name - for example, Home, Work, or Summer House.
6. Enter the address that should be the center of the geofence area.
7. Adjust the size of the area using the slider. A geofence must have a **minimum radius of 100 meters**.
8. Tap **Save** to activate your geofence.

Tip:

- You can add multiple geofences for the same watch - for example, one around home, work, or a summer house.
- Name your geofences clearly so you can easily tell them apart in the app.

Note:

If the watch temporarily loses its GPS signal, notifications that the user has left an area may be delayed until the connection is restored.

Related Content

- [What is a geofence?](#)