



[Knowledgebase](#) > [Setup and customization](#) > [How do I receive alarms?](#)

How do I receive alarms?

Mari Opstad - 2026-02-09 - [Comments \(0\)](#) - [Setup and customization](#)

How do I receive alarms?

To receive notifications and alerts from the app, such as when the watch leaves a geofence or when the battery is low, you must be added as an alarm receiver for the watch.

Step by step:

1. Log in to the Care Tracker app.
2. Select the watch you want to receive alarms from.
3. Select **Settings**.
4. Select **Alarm receivers/users**.
5. Enable yourself as an alarm receiver.

Once you have been added, you will receive alarms and notifications from the app.

Which alarms can I receive?

As an alarm receiver, you can receive notifications, for example when the watch moves outside a geofence or when the watch has a low battery.

Tip:

You can also add other people as alarm receivers by selecting **Add alarm receiver**.

Related Content

- [How do I add more alarmreceivers?](#)