



[Knowledgebase](#) > [Mobil app](#) > [Enheder](#) > [Device settings](#)

# Device settings

Mari Opstad - 2025-11-21 - [Comments \(0\)](#) - [Enheder](#)

## Device settings

This guide explains which settings you can adjust for a device in the Stella Care app, and how to customize sound and naming as needed.

### How to open device settings

1. Go to **Devices** in the app, or open the citizen's page and select the device.
2. Tap the device to open its details.
3. Tap **Settings** to open the device's settings menu.

### What you can change in device settings

#### Device Name

- You can change the device's name to make it easier to recognize in the app.
- Enter the new name and save the change.

#### Sound Settings

- Choose which sound the device should use for alarms.
- Choose which sound the device should use for warnings such as low battery.
- You can turn vibration and sounds on or off as needed.

These settings make it possible to tailor the device's notifications so they match the citizen's needs and your workflows.

### Tips

- Use a name that makes it easy to see which citizen the device belongs to.
- Choose a clear alarm sound so important events are not overlooked.

## Related Content

- [Notification Settings](#)
- [Account Settings \(edit name, email, phone\)](#)