



[Knowledgebase](#) > [Mobil app](#) > [Enheder](#) > [Play sound on the device](#)

Play sound on the device

Mari Opstad - 2025-11-19 - [Comments \(0\)](#) - [Enheder](#)

Play sound on the device

This guide explains how to make a GPS device play an audible sound from the Stella Care app. The function is used when you need to locate the device physically.

How to play sound on the device

1. Go to **Devices** in the app, or open the citizen's page and select the device.
2. Tap the device to open its details.
3. Tap **Make noise**.
4. The device will now emit a clear sound for approximately **20 seconds**.

This makes it easier to find the device if it is hidden in a bag, bed, jacket, or in a room where it is not visible.

Tips

- If you are near the citizen, the sound can help you locate the device quickly without needing to start a call.

Related Content

- [Call a device](#)
- [Track a device in the app](#)